




# Contour<sup>®</sup> plus ONE<sup>))</sup>

Blood Glucose  
Monitoring System

# Setting up the **CONTOUR™ DIABETES** app

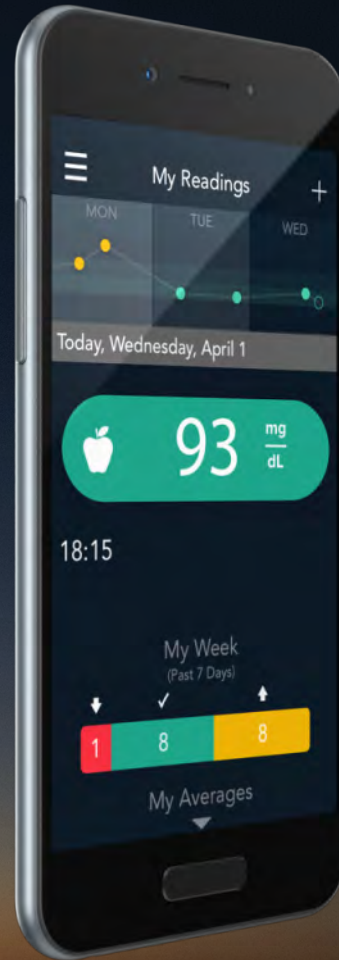


**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System

## Setup Process

The first time you use the **CONTOUR DIABETES app**, you will be asked to complete the Setup process to customize your settings.

Once you have created your account and paired a meter to your mobile device you will then be taken through the Setup process.

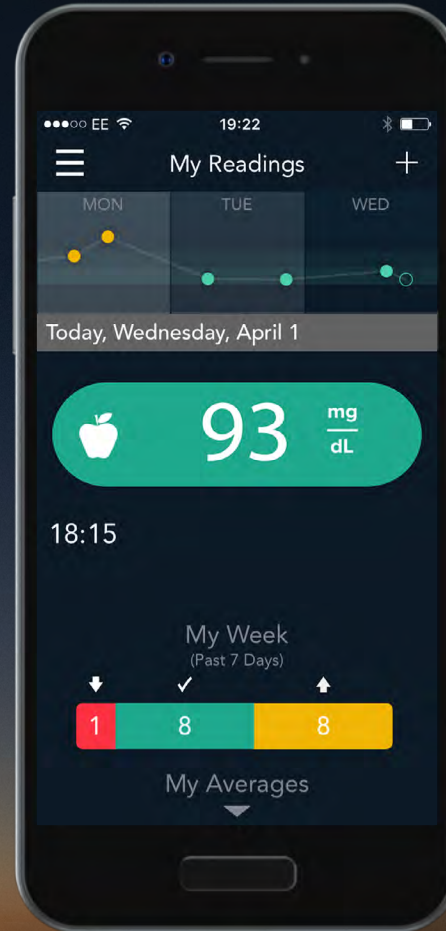


Contour<sup>®</sup>  
plus **ONE**<sup>»</sup>  
Blood Glucose  
Monitoring System

1.

## Time and Date

The **CONTOUR DIABETES app** automatically syncs your meter Time and Date with your mobile device to keep your meter time accurate even when you change time zones and during daylight saving time.

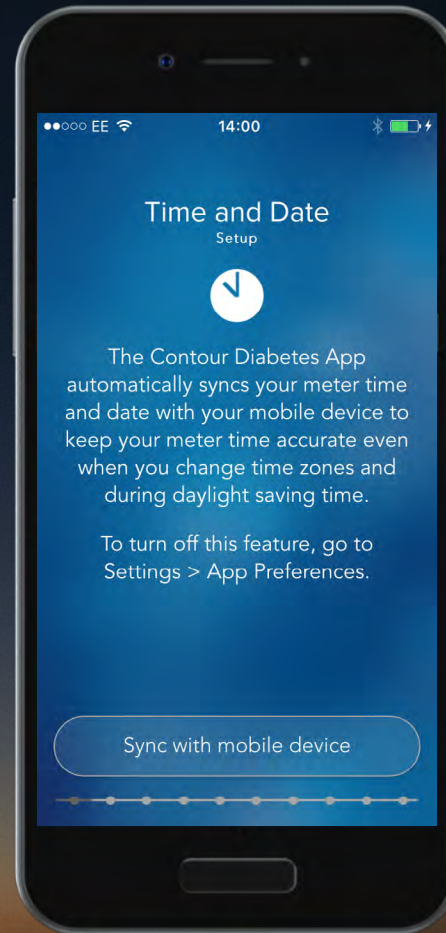


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

2.

## Time and Date

Select 'Sync with mobile device' to set your local Time and Date for accurate time stamps on your blood sugar readings.

Select 'Continue'.

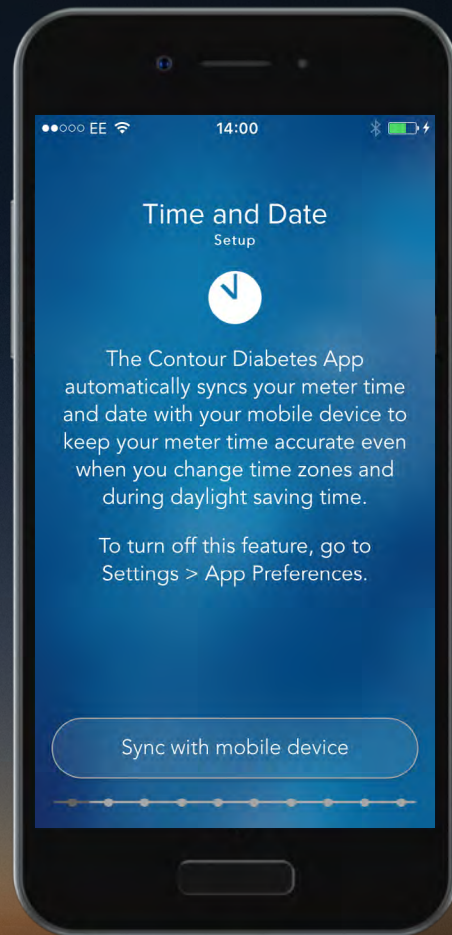


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

2.

## Time and Date

Select 'Sync with mobile device' to set your local Time and Date for accurate time stamps on your blood sugar readings.

Select 'Continue'.

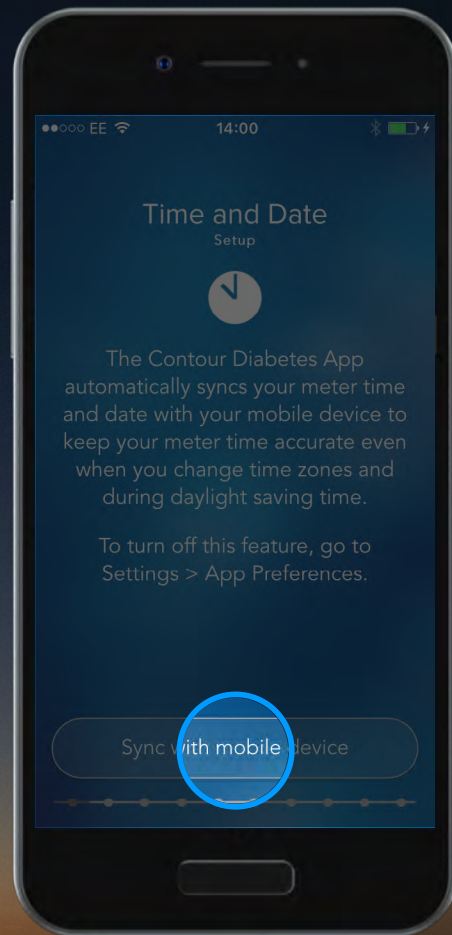


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

2.

## Time and Date

Select 'Sync with mobile device' to set your local Time and Date for accurate time stamps on your blood sugar readings.

Select 'Continue'.

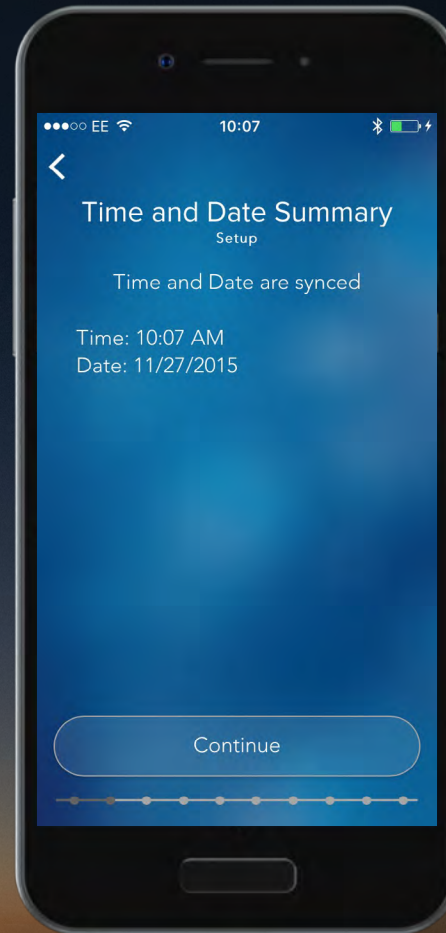


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

2.

## Time and Date

Select 'Sync with mobile device' to set your local Time and Date for accurate time stamps on your blood sugar readings.

Select 'Continue'.

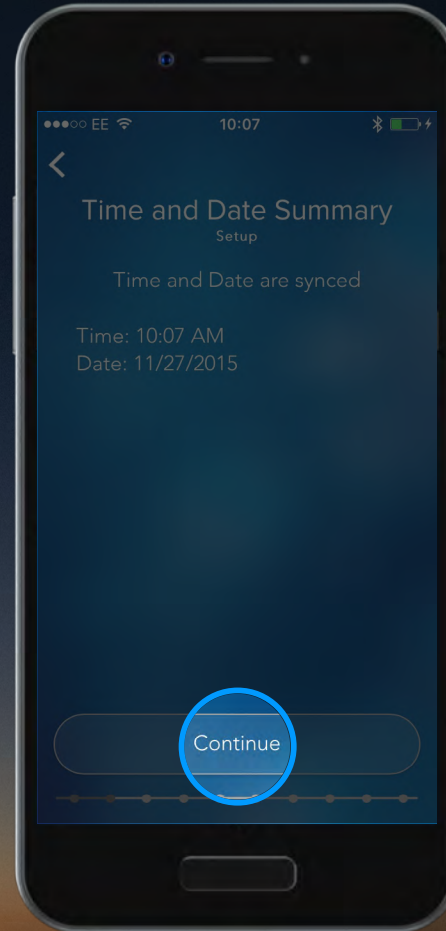

Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System




3.

### Day Dividers

The **CONTOUR DIABETES app** uses Day Dividers to divide your day into meal times and overnight to organize your blood glucose readings.

Select '**Continue**' to set your Day Dividers.

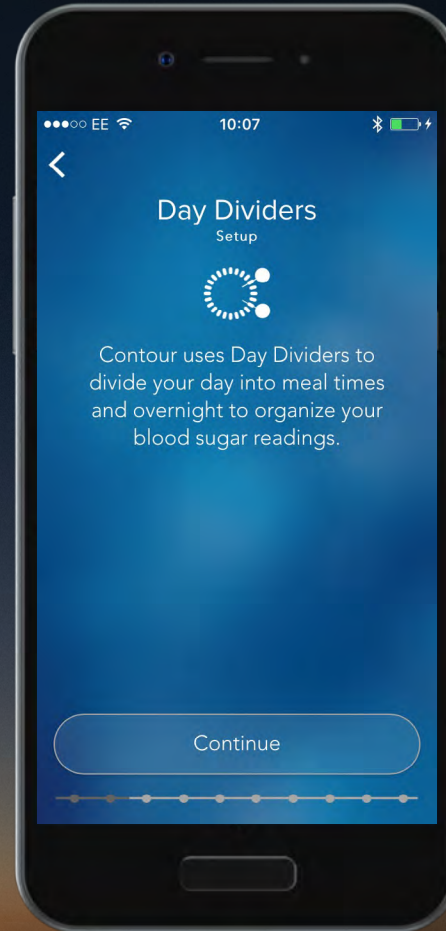

Contour<sup>®</sup>  
plus **ONE** »  
Blood Glucose  
Monitoring System

3.

### Day Dividers

The **CONTOUR DIABETES app** uses Day Dividers to divide your day into meal times and overnight to organize your blood glucose readings.

Select '**Continue**' to set your Day Dividers.




Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

4.

## Day Dividers

You can accept the suggested times by selecting 'Accept'.

Tap the 'arrow' next to each divider to view or change it.

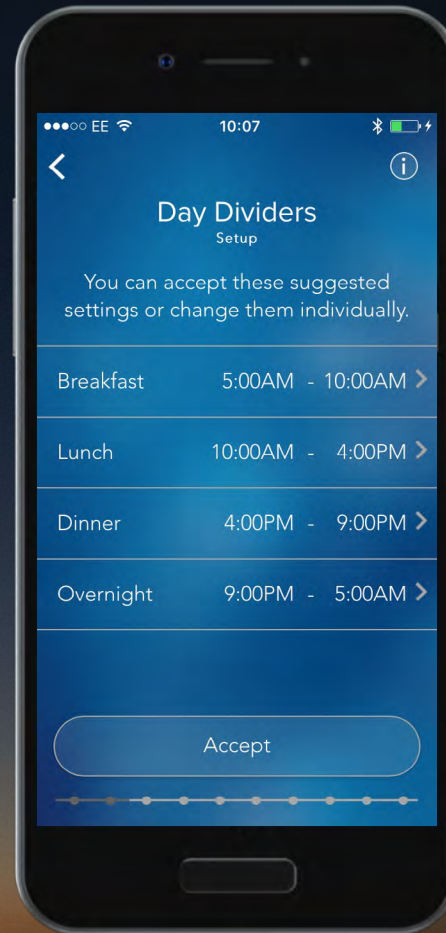


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

4.

## Day Dividers

You can accept the suggested times by selecting 'Accept'.

Tap the 'arrow' next to each divider to view or change it.

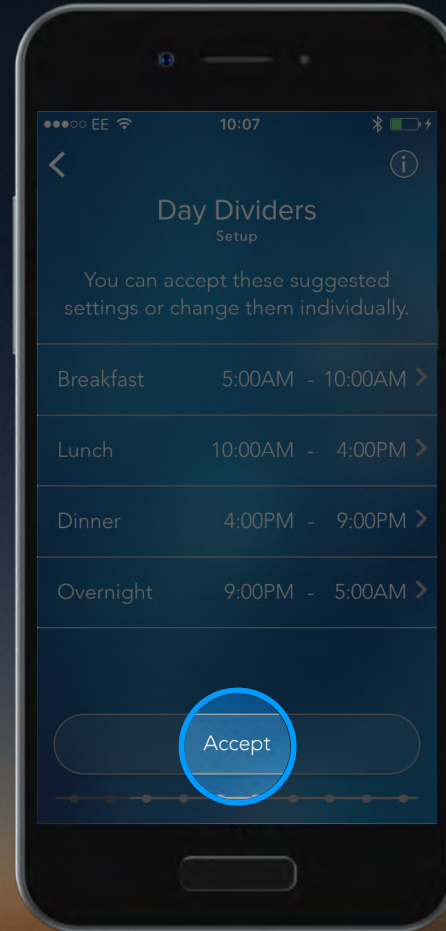


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

4.

## Day Dividers

You can accept the suggested times by selecting 'Accept'.

Tap the 'arrow' next to each divider to view or change it.

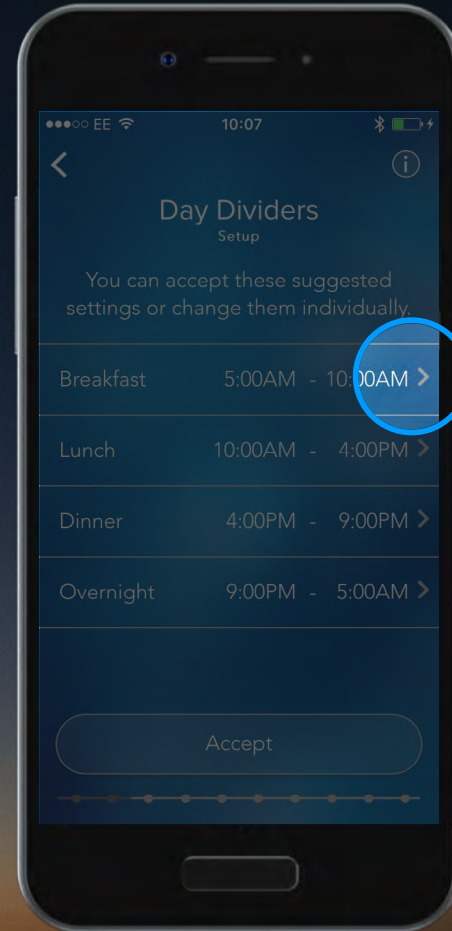


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

5.

## Day Dividers

Slide the time marker around the clock face to change the time.

Select 'Done' when you have finished.

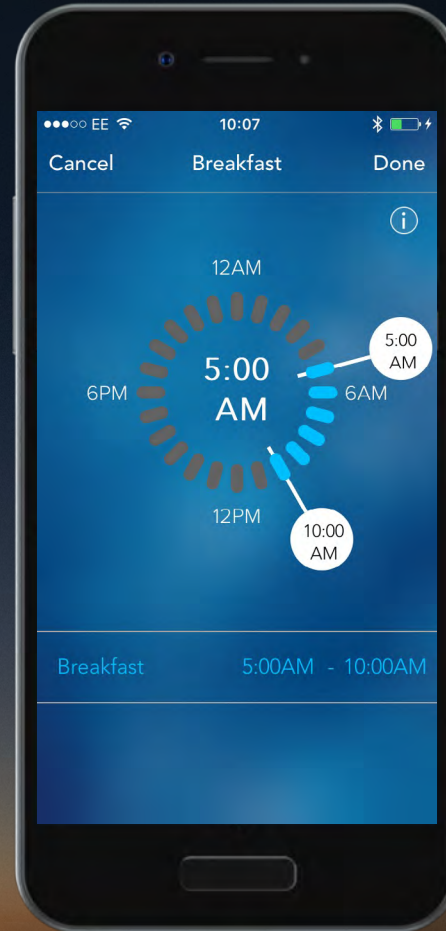


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

5.

## Day Dividers

Slide the time marker around the clock face to change the time.

Select 'Done' when you have finished.

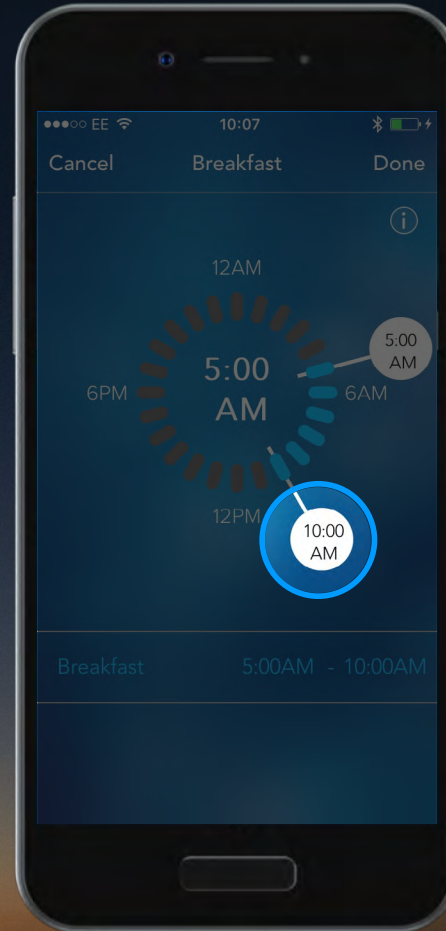


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

5.

## Day Dividers

Slide the time marker around the clock face to change the time.

Select 'Done' when you have finished.

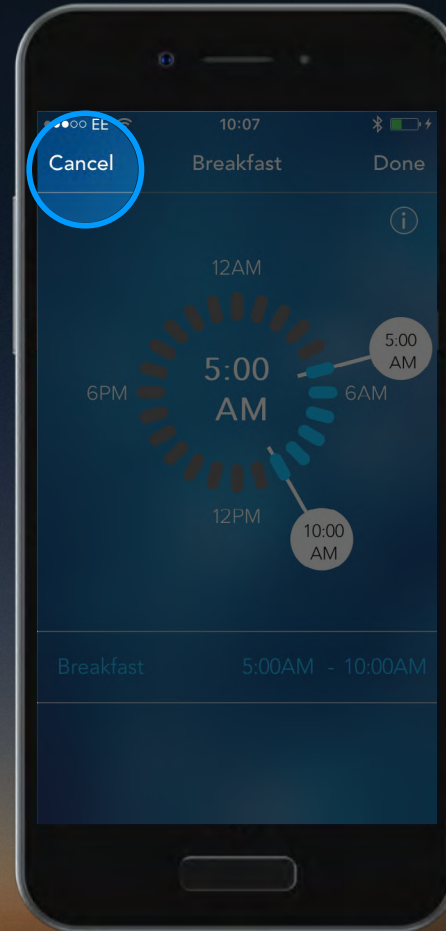


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



6.

## Day Dividers

Select 'Accept' to confirm your chosen timings.

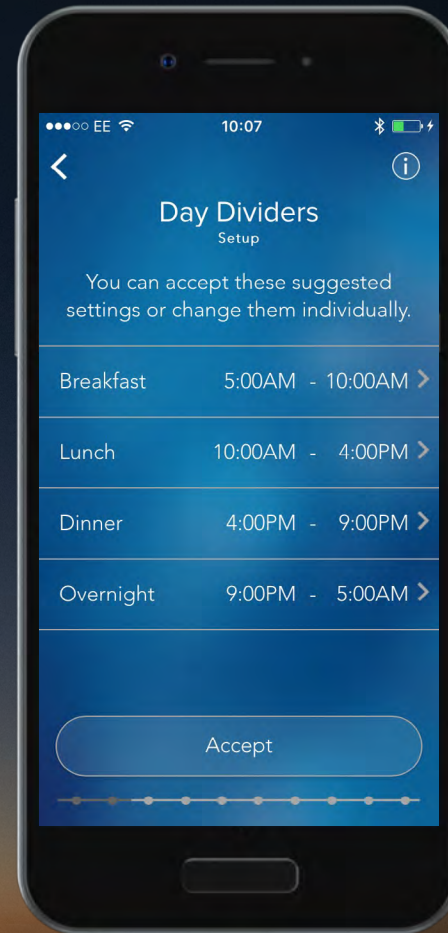


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

6.

## Day Dividers

Select 'Accept' to confirm your chosen timings.

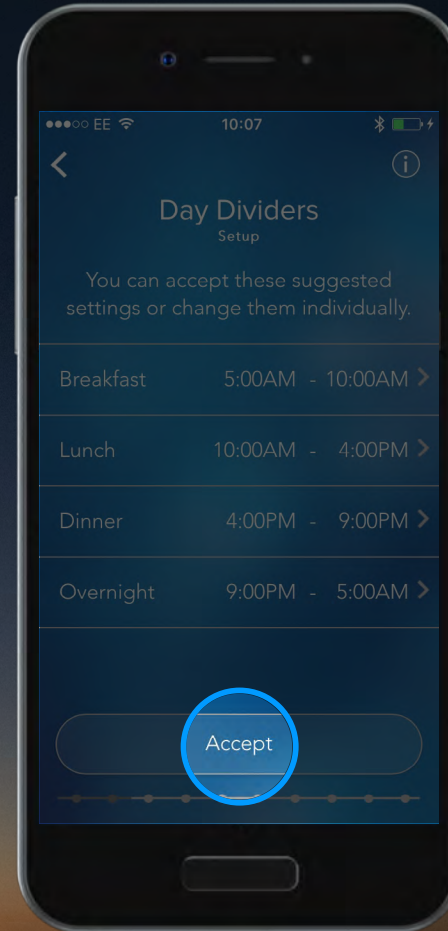


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

7.

## Meal Markers

The **CONTOUR DIABETES app** uses Meal Markers to note when a blood glucose reading is taken in relation to a meal.

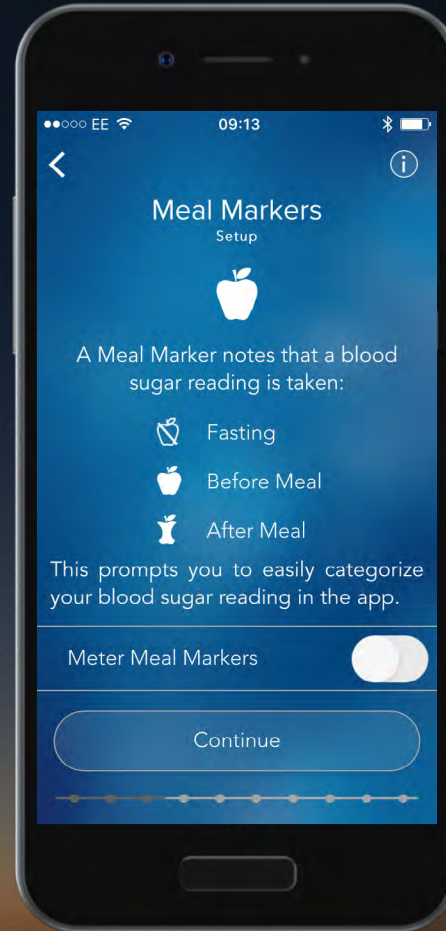


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

8.

## Meal Markers

You can always categorize your blood glucose readings with meal markers in the app.

If you prefer to meal mark on your meter after each test, select the **'toggle button'** to turn meal markers on and then select **'Continue'**.

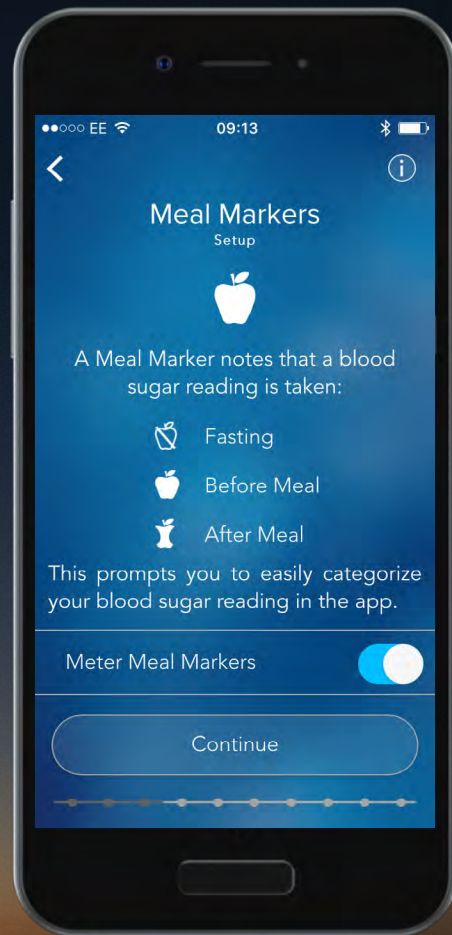


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

8.

## Meal Markers

You can always categorize your blood glucose readings with meal markers in the app.

If you prefer to meal mark on your meter after each test, select the **'toggle button'** to turn meal markers on and then select **'Continue'**.

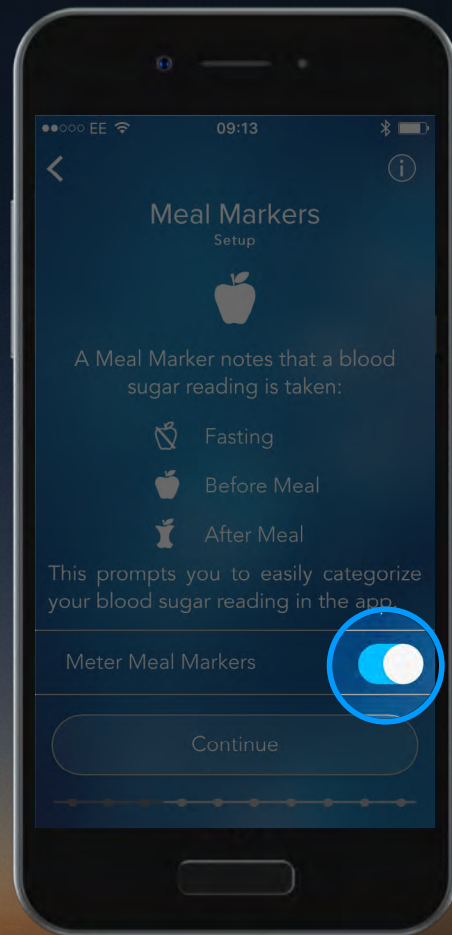


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

8.

## Meal Markers

You can always categorize your blood glucose readings with meal markers in the app.

If you prefer to meal mark on your meter after each test, select the **'toggle button'** to turn meal markers on and then select **'Continue'**.

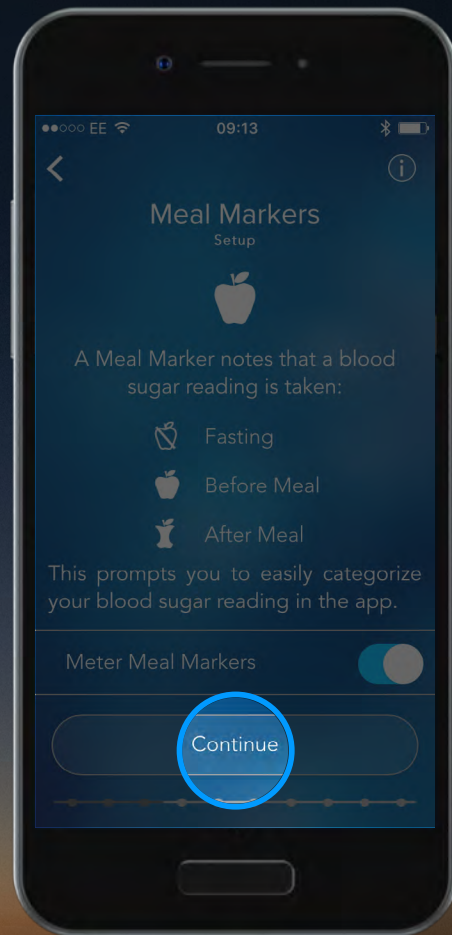

Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System


9.

## Target Ranges

The **CONTOUR DIABETES app** compares your blood glucose reading with a Target Range to let you know if it is above, within or below target.

Select '**Continue**' to set your Target Ranges.

**WARNING** - Talk to your Health Care Professional before setting any Target Ranges in the **CONTOUR DIABETES app**.

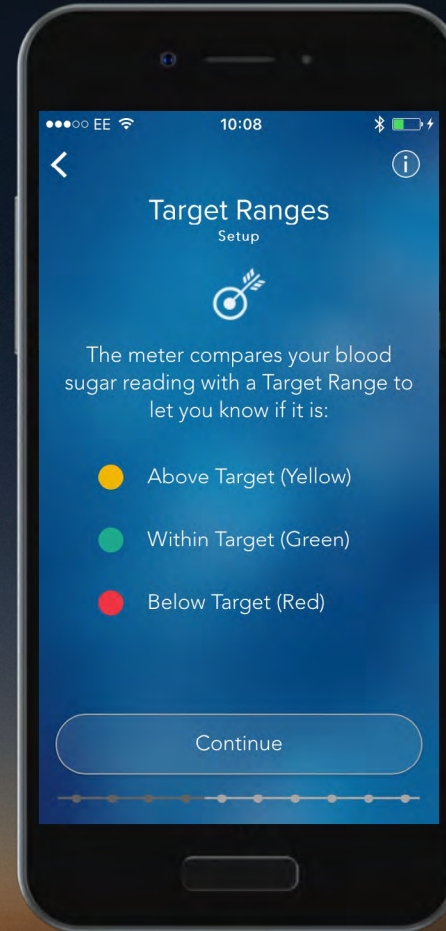

**Contour<sup>®</sup>**  
**plus ONE<sup>™</sup>**  
Blood Glucose  
Monitoring System


9.

## Target Ranges

The **CONTOUR DIABETES app** compares your blood glucose reading with a Target Range to let you know if it is above, within or below target.

Select '**Continue**' to set your Target Ranges.

**WARNING** - Talk to your Health Care Professional before setting any Target Ranges in the **CONTOUR DIABETES app**.

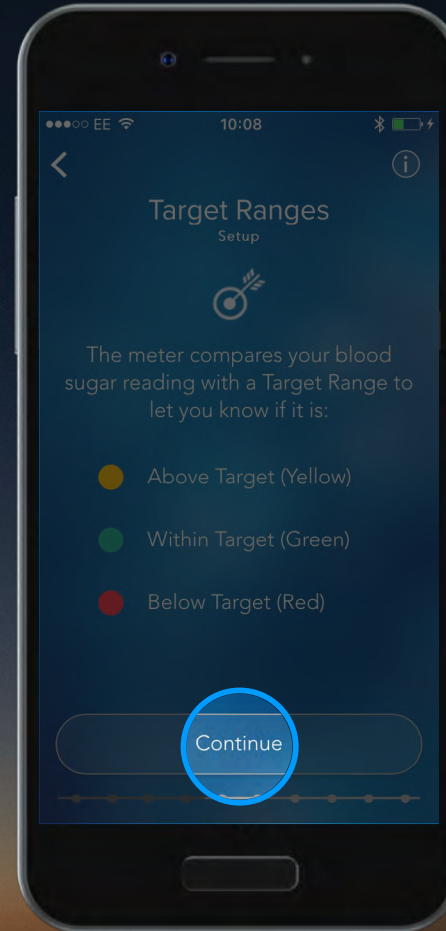

Contour<sup>®</sup>  
plus **ONE**<sup>»</sup>  
Blood Glucose  
Monitoring System




9.

## Target Ranges

The **CONTOUR DIABETES app** compares your blood glucose reading with a Target Range to let you know if it is above, within or below target.

Select '**Continue**' to set your Target Ranges.

**WARNING** - Talk to your Health Care Professional before setting any Target Ranges in the **CONTOUR DIABETES app**.

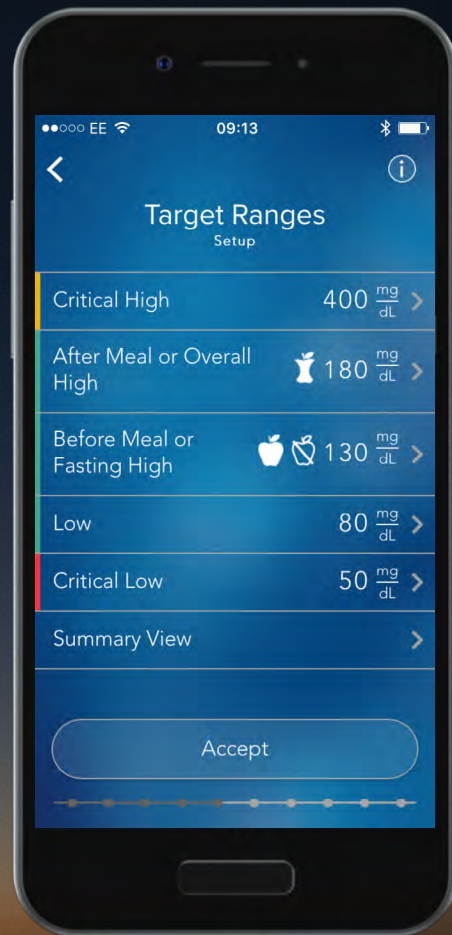

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System

10.

## Target Ranges

You can accept the pre-set ranges by selecting 'Accept'.

Tap the 'arrow' next to each divider to view or change it.



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

10.

## Target Ranges

You can accept the pre-set ranges by selecting 'Accept'.

Tap the 'arrow' next to each divider to view or change it.



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

11.

## Target Ranges

Slide the target value up and down to change a Target Range, or accept the pre-set Target Ranges.



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

11.

## Target Ranges

Slide the target value up and down to change a Target Range, or accept the pre-set Target Ranges.



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

12.

## Target Ranges

Select 'Accept' to set your Target Ranges.



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

12.

## Target Ranges

Select 'Accept' to set your Target Ranges.




Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

13.

## Target Lights

The **CONTOUR® PLUS ONE** smart meter test strip port displays a colored light representing your blood glucose reading compared to a Target Range.

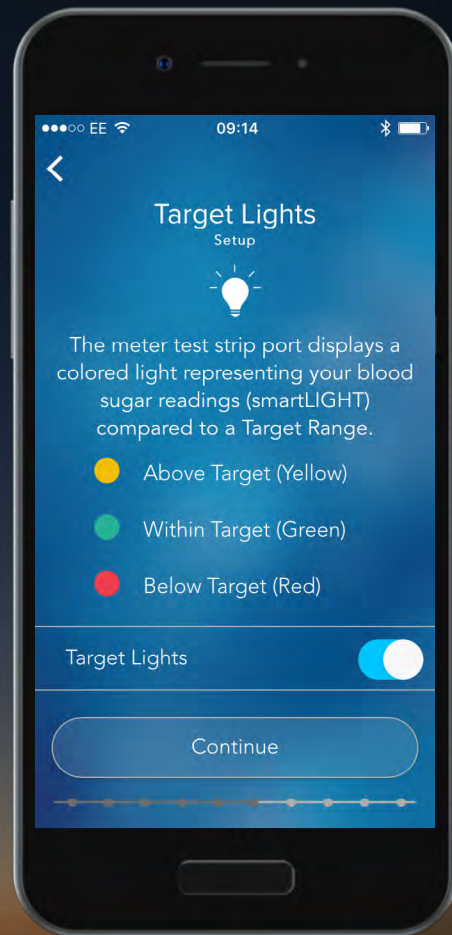

Contour®  
plus ONE»  
Blood Glucose  
Monitoring System




14.

## Target Lights

Tap the toggle button to turn Target Lights on and off.

Select 'Continue' to set.

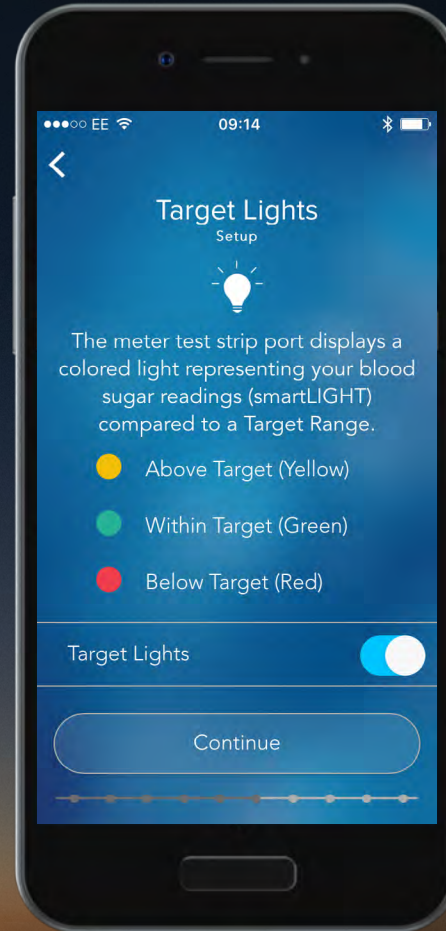


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

14.

## Target Lights

Tap the toggle button to turn Target Lights on and off.

Select 'Continue' to set.

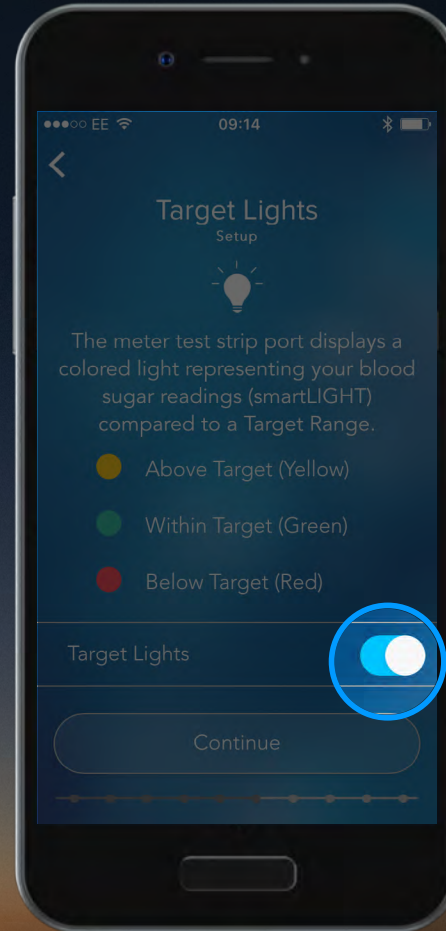


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

14.

## Target Lights

Tap the toggle button to turn Target Lights on and off.

Select 'Continue' to set.

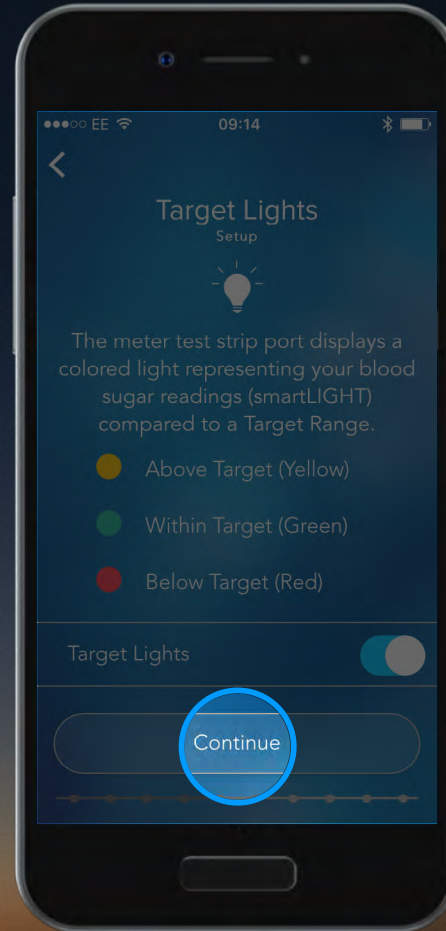


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

15.

## Meter Setup Summary

Once you have finished the initial meter set up, a summary screen appears.

Select 'Continue' to confirm, or tap the toggle buttons to turn meter settings on or off.

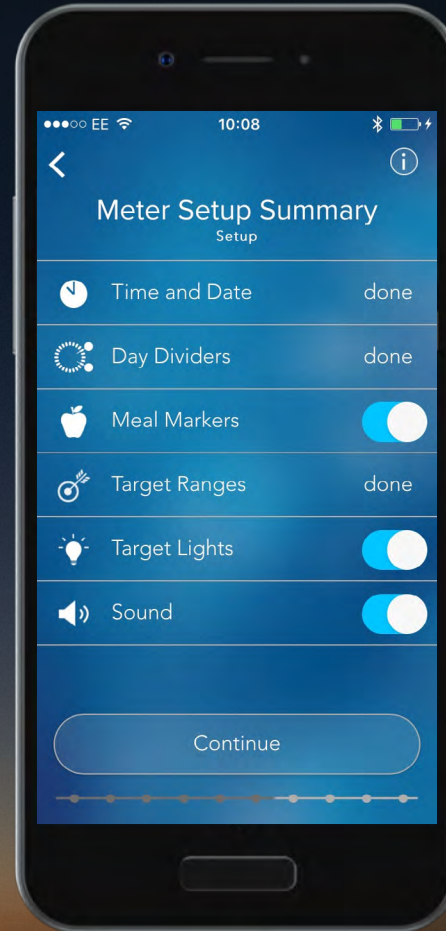


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

15.

## Meter Setup Summary

Once you have finished the initial meter set up, a summary screen appears.

Select 'Continue' to confirm, or tap the toggle buttons to turn meter settings on or off.

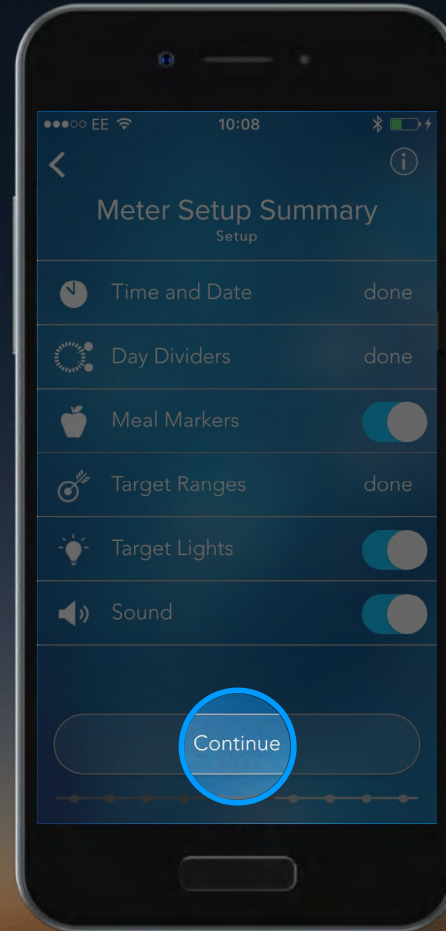


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

16.

## Meter Setup Summary

Now select 'Continue' to complete your **CONTOUR DIABETES app** setup.

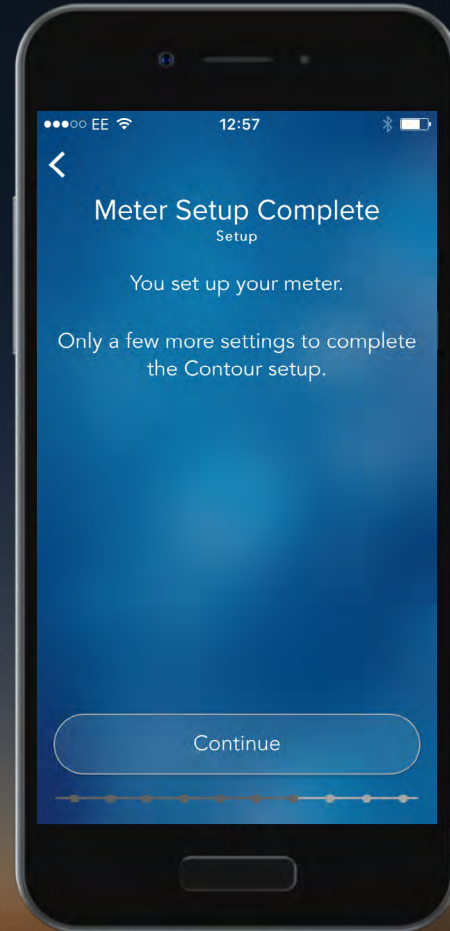


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

16.

## Meter Setup Summary

Now select 'Continue' to complete your **CONTOUR DIABETES app** setup.

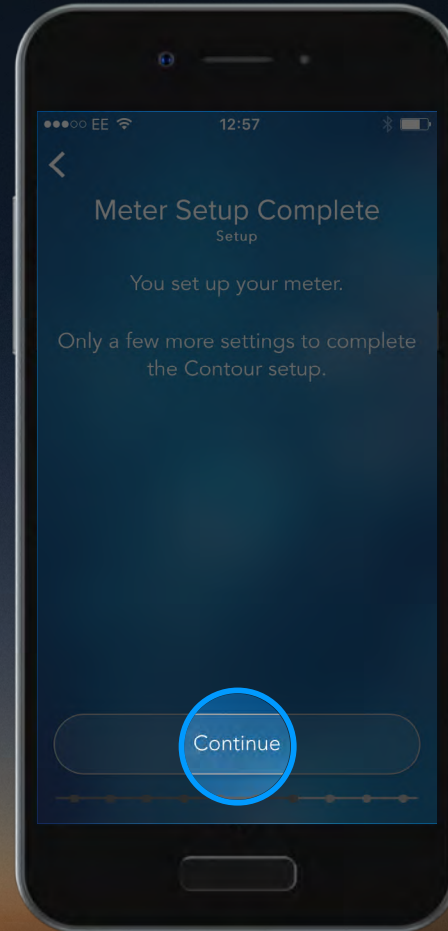


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

17.

## Emergency Calls

The **CONTOUR DIABETES app** prompts you to make an emergency call in the case of critically high or critically low blood glucose levels.

Select '**Continue**' to set up your local emergency number.

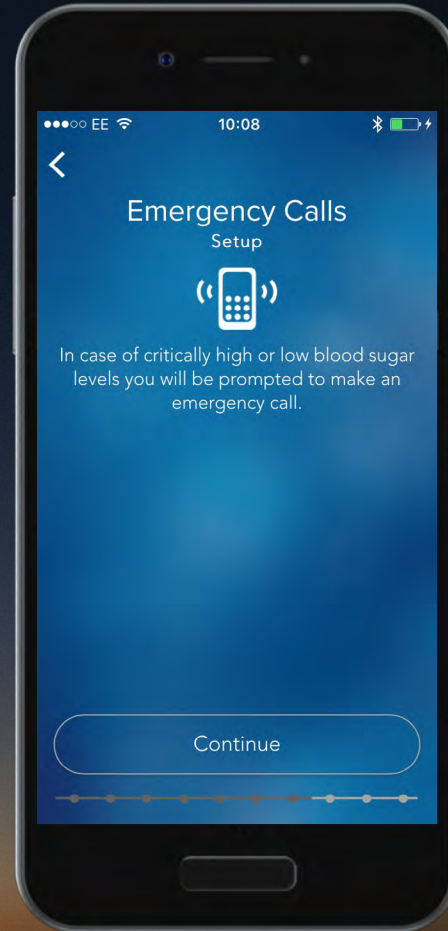

Contour<sup>®</sup>  
plus **ONE**<sup>™</sup>  
Blood Glucose  
Monitoring System




17.

## Emergency Calls

The **CONTOUR DIABETES app** prompts you to make an emergency call in the case of critically high or critically low blood glucose levels.

Select '**Continue**' to set up your local emergency number.

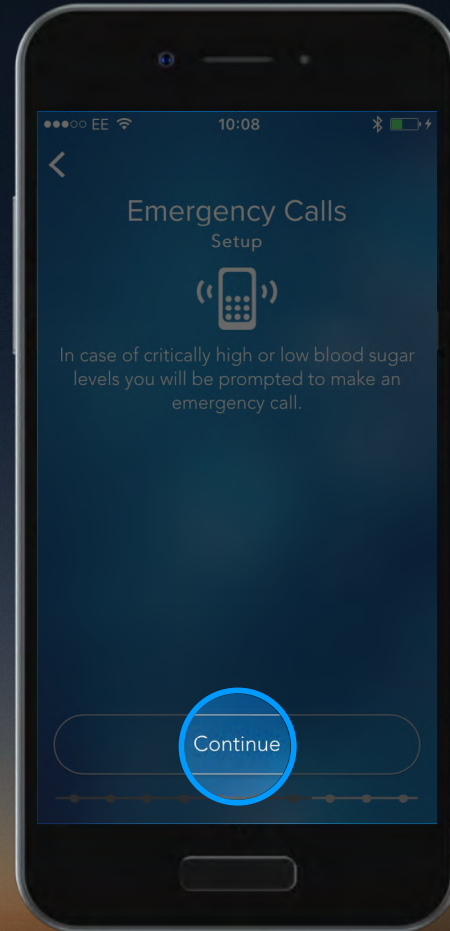


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System

18.

## Emergency Calls

Select your local emergency number using the keyboard and select 'Done' when complete.

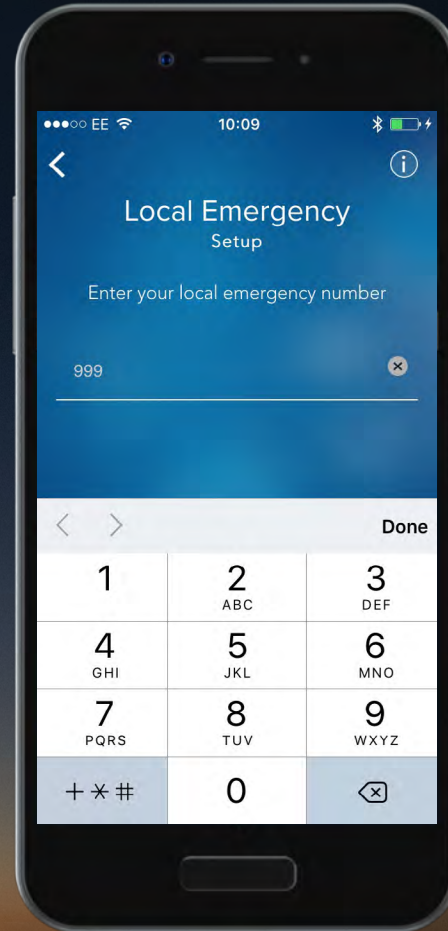


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

18.

## Emergency Calls

Select your local emergency number using the keyboard and select 'Done' when complete.

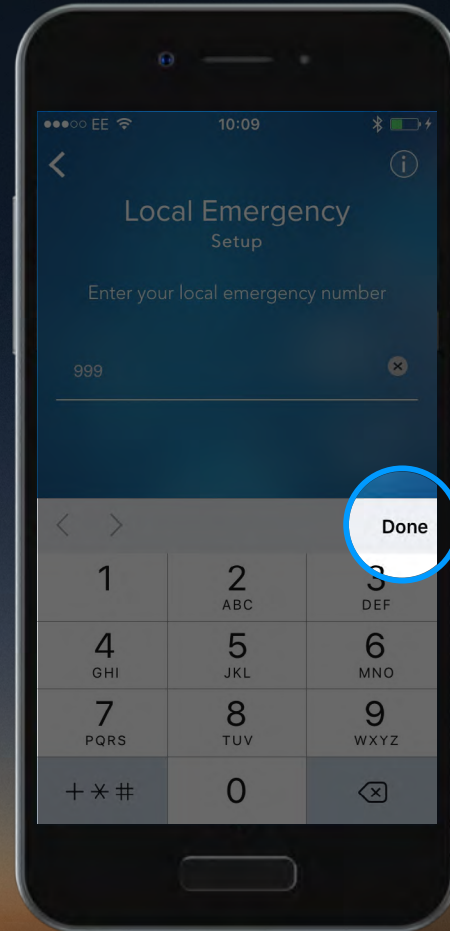


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

19.

## Emergency Contacts

The **CONTOUR DIABETES app** displays your Emergency Contacts when your blood glucose reaches a critical high or low level.

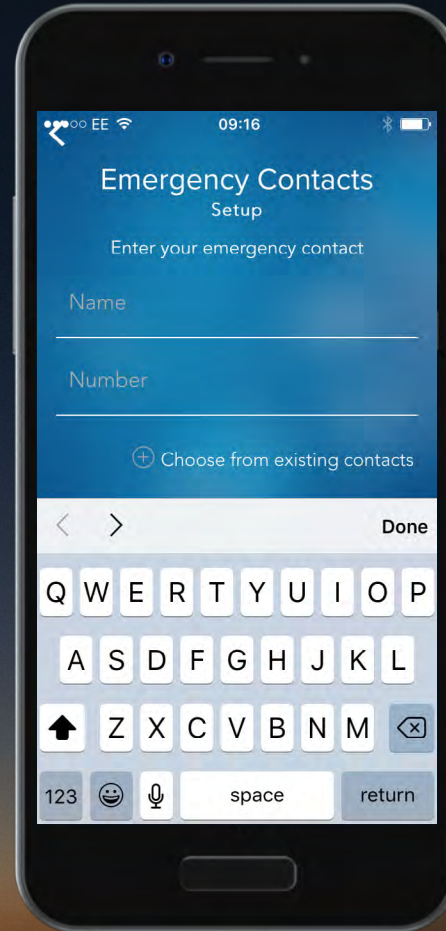


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

20.

## Emergency Contacts

Enter the name and number of your Emergency Contact using the keyboard.

Select 'Done' when complete.

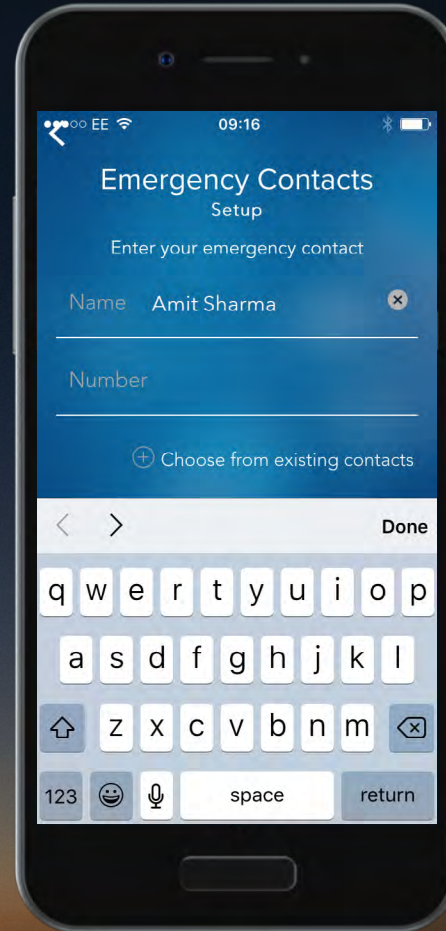


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

20.

## Emergency Contacts

Enter the name and number of your Emergency Contact using the keyboard.

Select 'Done' when complete.

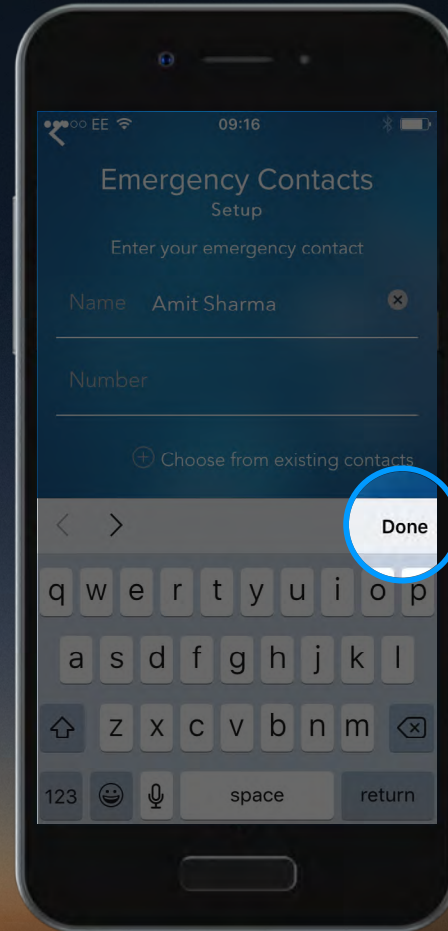


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

21.

## Emergency Contacts

You will be shown an overview of your entries.

Select 'Continue' if all are correct.

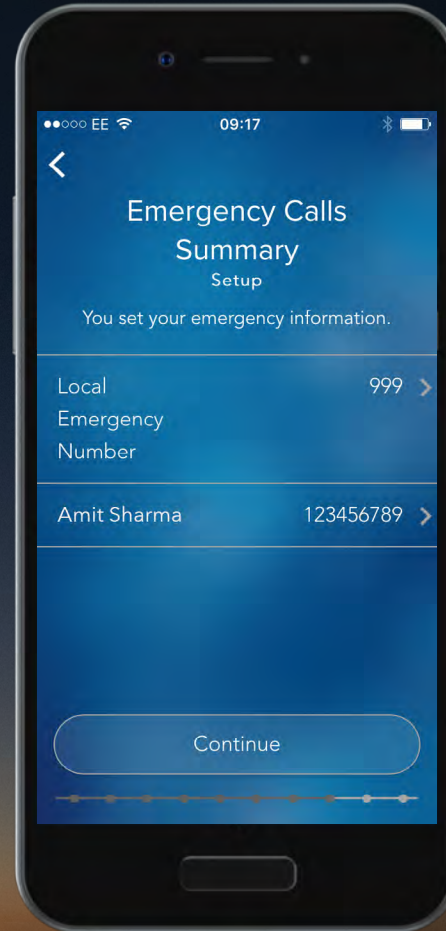


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

21.

## Emergency Contacts

You will be shown an overview of your entries.

Select 'Continue' if all are correct.

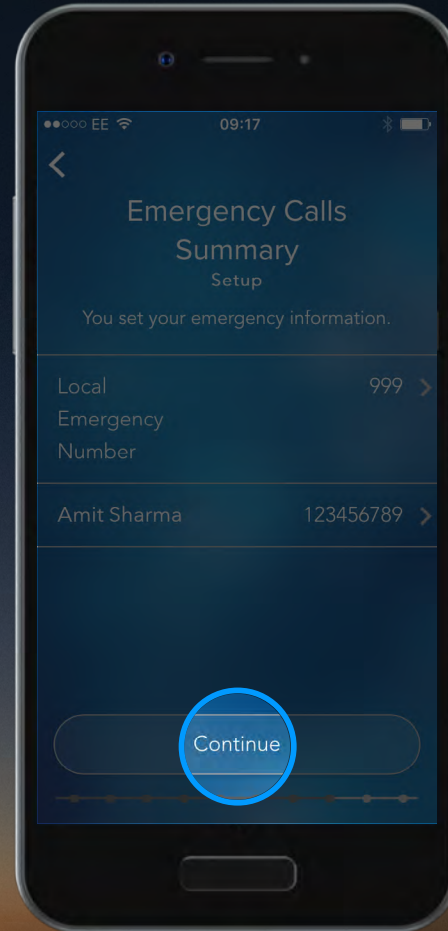


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



22.

## Profile

To help personalize information and improve your overall experience, set up your profile by selecting 'Set up profile'.

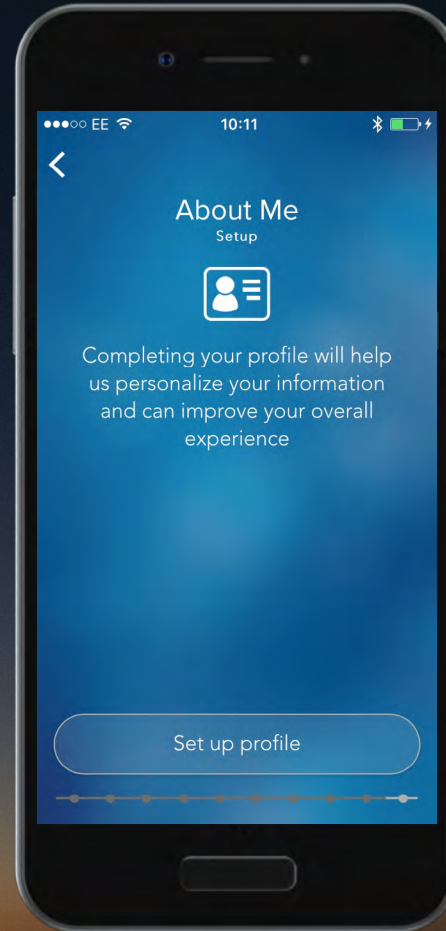


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

22.

## Profile

To help personalize information and improve your overall experience, set up your profile by selecting 'Set up profile'.

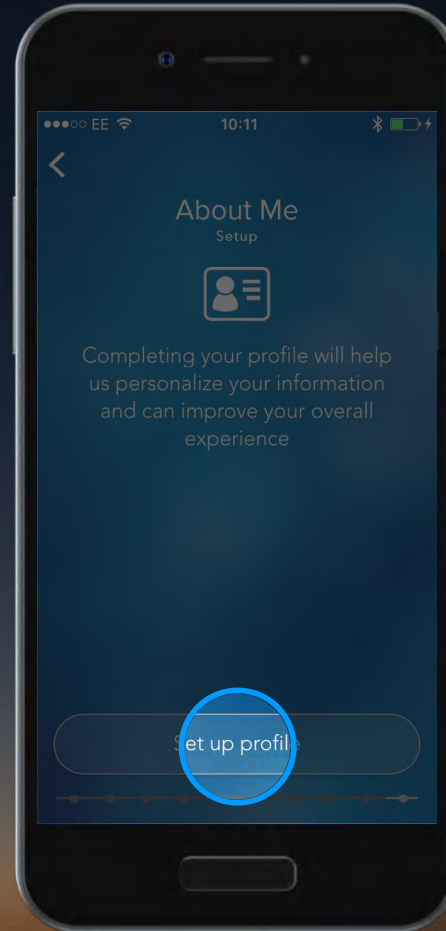


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

23.

## Profile

Enter your date of birth by sliding the dates up and down and select 'Continue'.

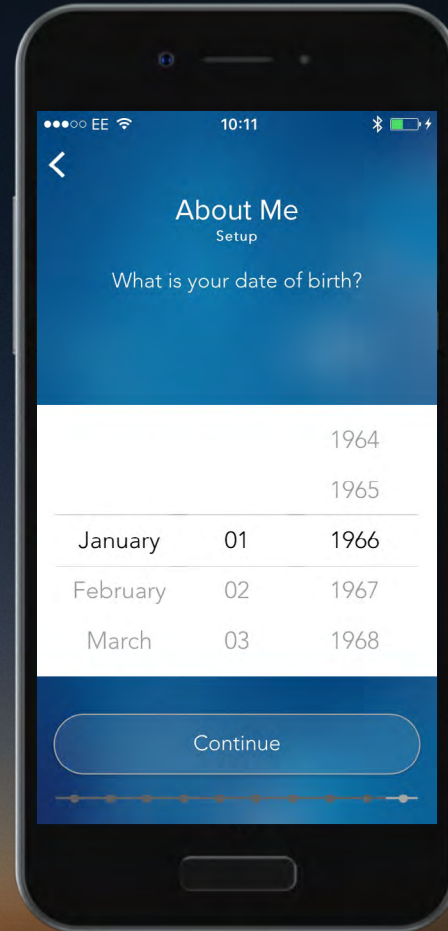


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

diabetes.ascensia.com

23.

## Profile

Enter your date of birth by sliding the dates up and down and select 'Continue'.

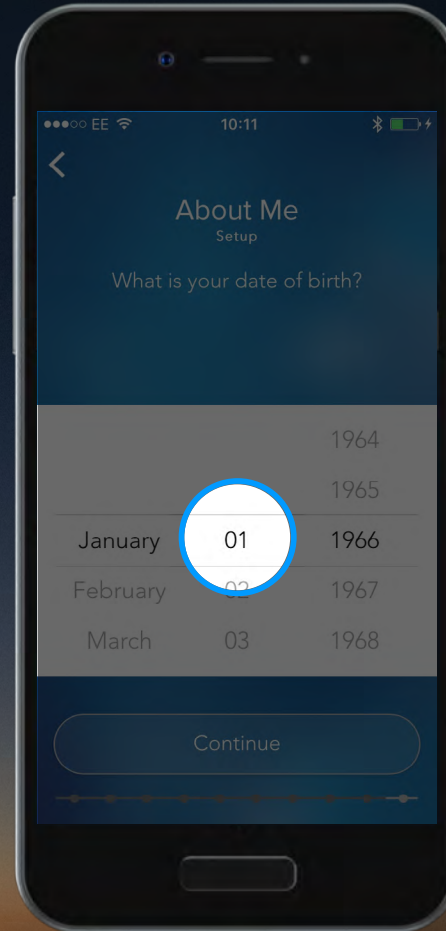


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

23.

## Profile

Enter your date of birth by sliding the dates up and down and select 'Continue'.

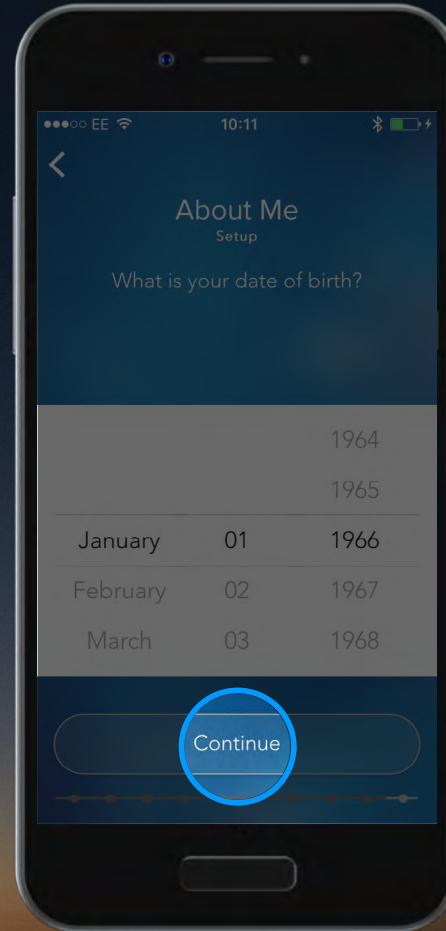

Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

24.

## Profile

Select your gender and 'Continue'.





Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

24.

## Profile

Select your gender and 'Continue'.

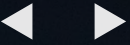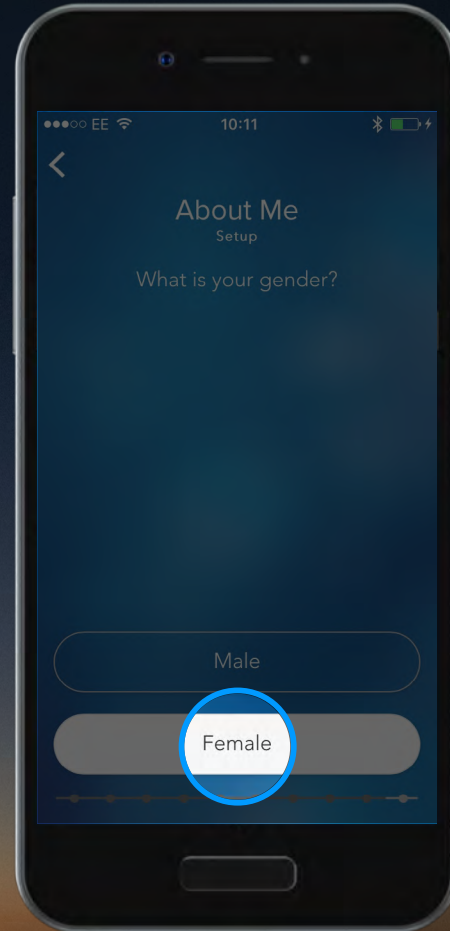


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

25.

## Profile

Select the type of diabetes you have.

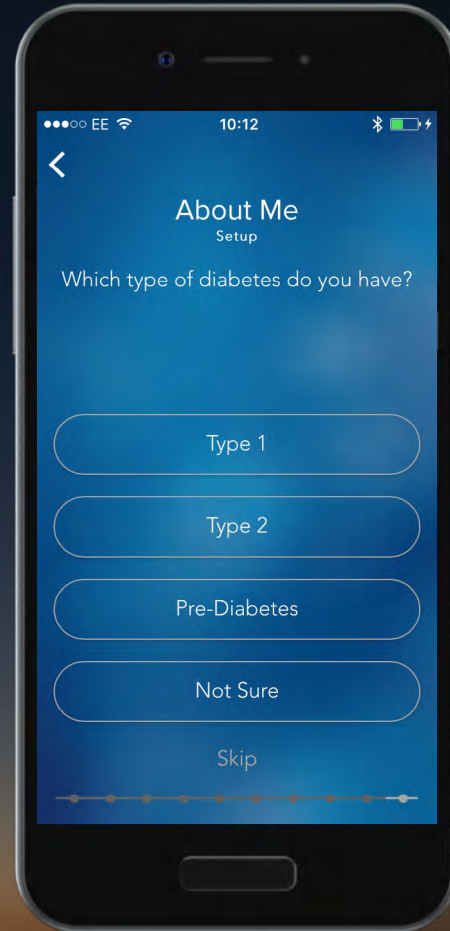


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



25.

## Profile

Select the type of diabetes you have.

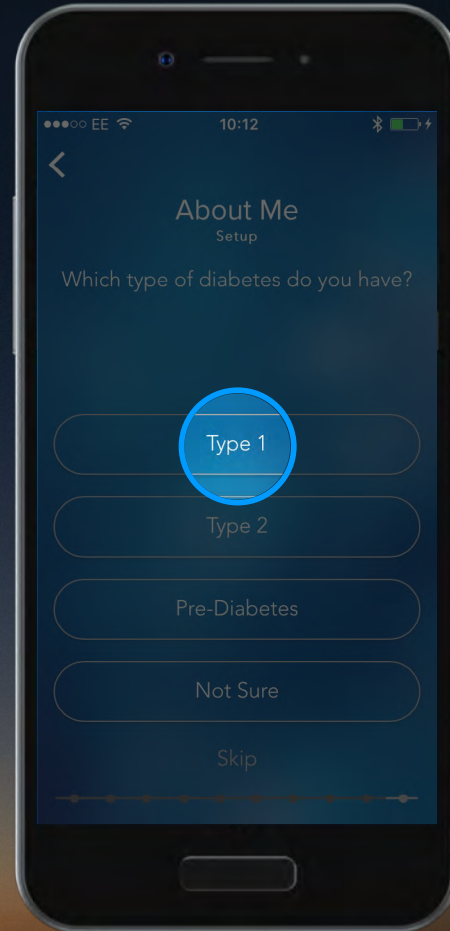


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

26.

## Profile

Now select your current therapy.

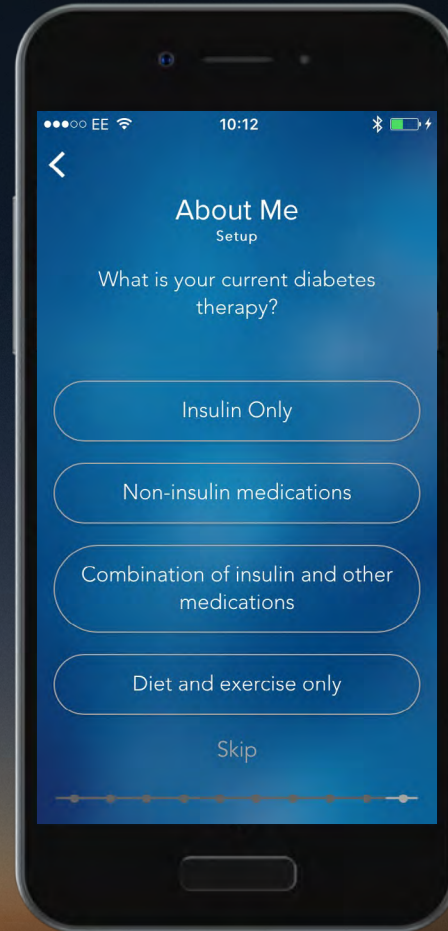


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

26.

## Profile

Now select your current therapy.

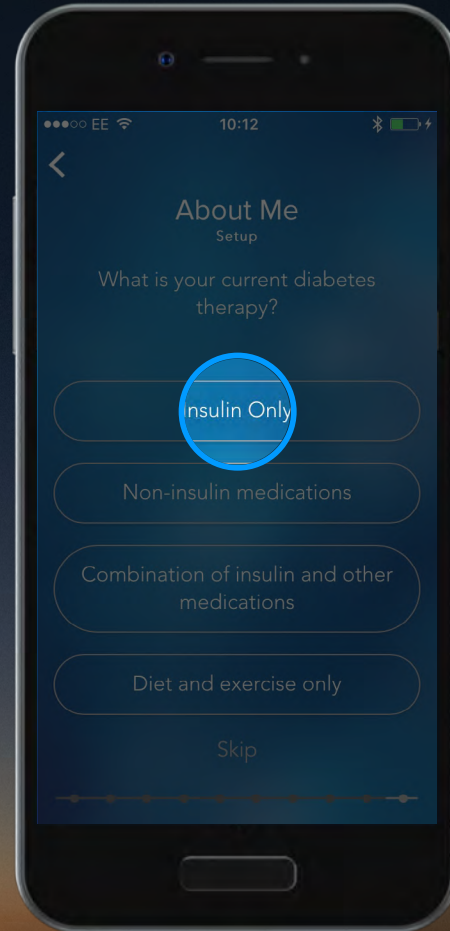


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

27.

## Profile

Your Profile is now complete, select 'Continue'.

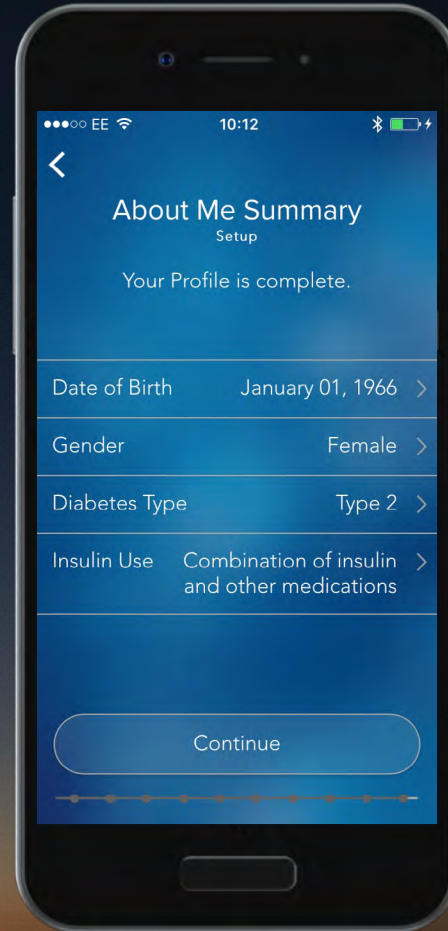


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

27.

## Profile

Your Profile is now complete, select 'Continue'.

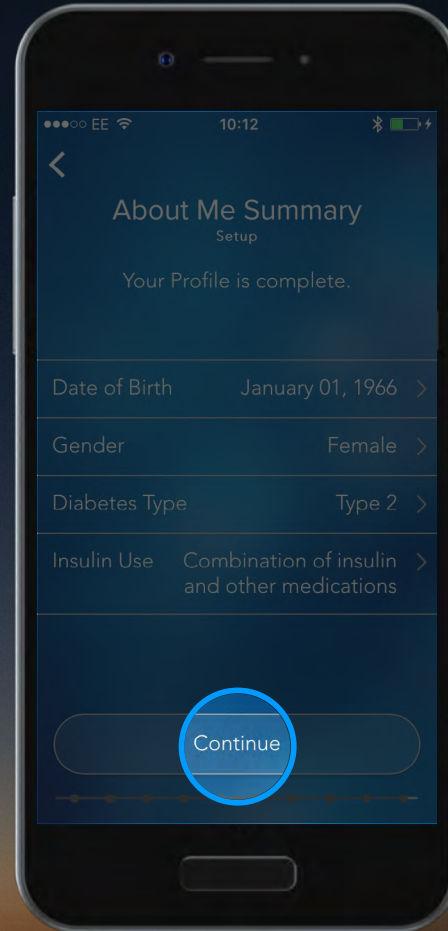


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

28.

## Setup Complete.

Your **CONTOUR DIABETES app** Setup is now complete.

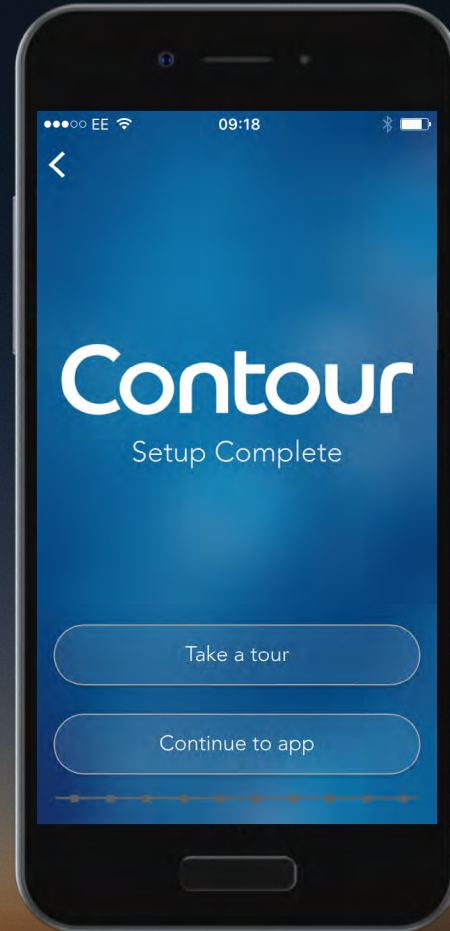


Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

29.

## Settings

You can change any of these settings by selecting 'Settings' from the Main Menu.

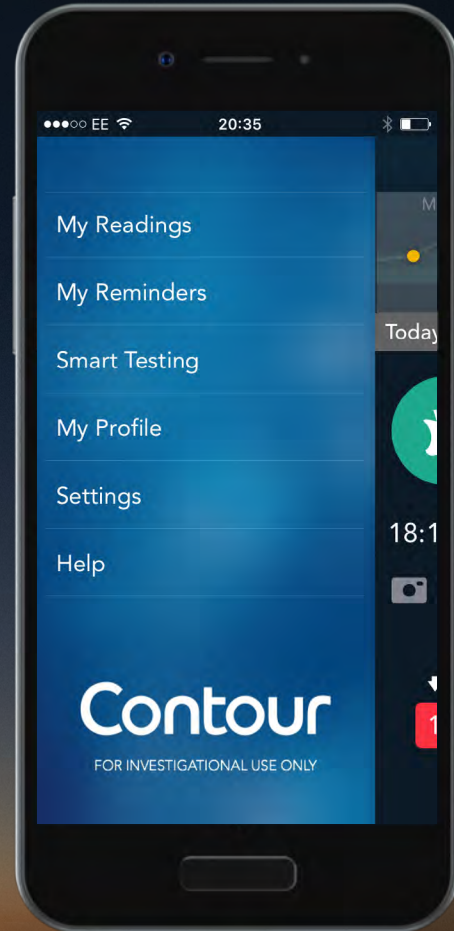


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

29.

## Settings

You can change any of these settings by selecting 'Settings' from the Main Menu.

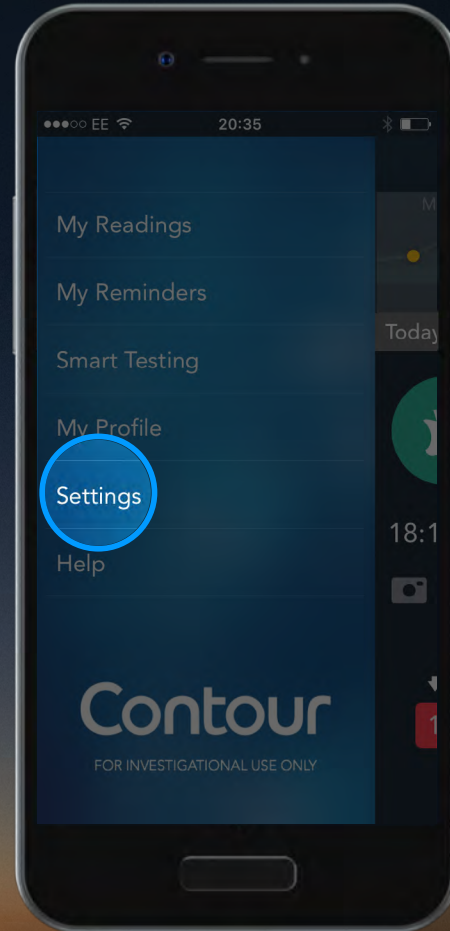


Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



30.

## Settings

From the **Settings** menu, you can view and change details about your Meter, your Account, Target Ranges, Day Dividers, and more.

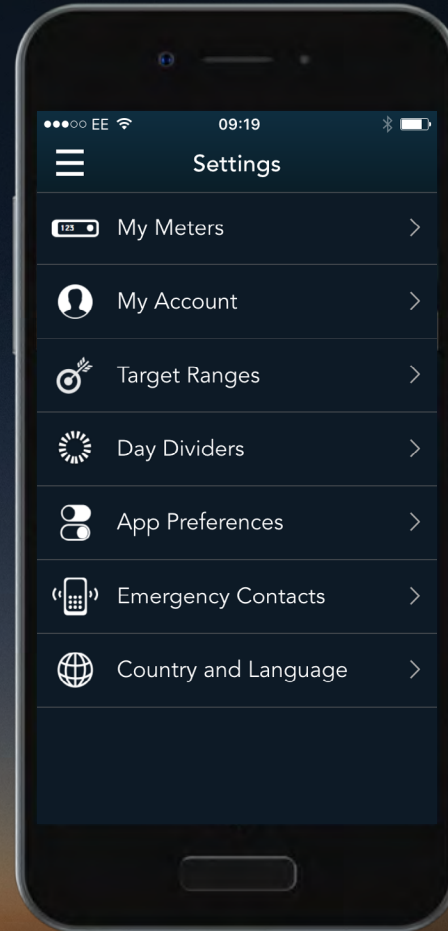

Contour<sup>®</sup>  
plus **ONE**<sup>™</sup>  
Blood Glucose  
Monitoring System

Talk to your Healthcare Professional and refer to the product user guide if you have questions about the **CONTOUR®PLUS ONE** smart meter and the **CONTOUR™DIABETES app**. For more information, and to find out whether the **CONTOUR®PLUS ONE** meter is available in your country, visit [diabetes.ascensia.com](http://diabetes.ascensia.com)




Get the app



Apple and the Apple logo are trademarks of Apple Inc., registered in the US and other countries. App Store is a service mark of Apple Inc. Google Play is a trademark of Google Inc.

Ascensia, the Ascensia Diabetes Care logo and CONTOUR are trademarks of Ascensia Diabetes Care Holdings AG.

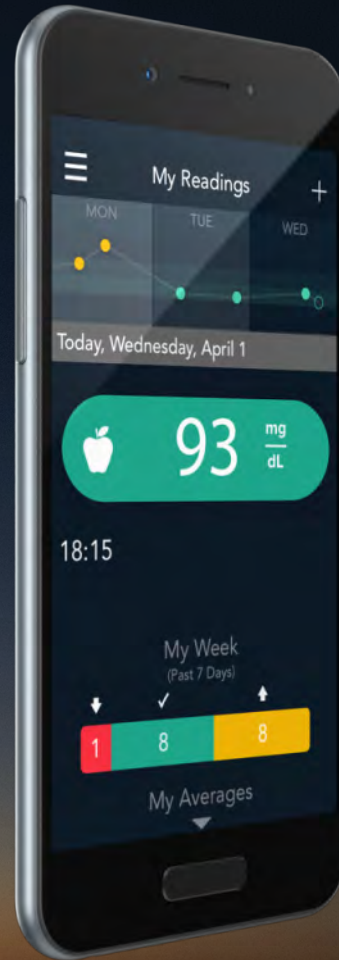

Contour  
plus **ONE**  
Blood Glucose  
Monitoring System



# Contour<sup>®</sup>

diabetes solutions